

# RMS Knee Adductors

Jan 2018

(Evazote Pads part numbers L7-P01-S25/50/75 and L7-P01-L25/50/75)  
(Gel Pads part numbers L7-P02-S25/50/75 and L7-P02-L25/50/75)



**!** RMS Knee Adductor Pads have been designed to assist with the positioning of an occupant's legs when seated in their wheelchair.

The mounting brackets are fully adjustable, allowing the pads to be positioned correctly at the knee, for optimum support.

Any reference to left or right relates to the left or right-hand side of the wheelchair as viewed from a seated position in the wheelchair.


RMS recommend fitment should be done by a suitably qualified person.

Before commencing, please ensure you have the correct size of Mounting Clamp Fig.3, to suit the wheelchair hanger size.

**IMPORTANT NOTE:** To assist with ease of transferring, the wheelchair footrest hangers, complete with attached Adductors, should be fully swung-away or removed if necessary, during user transfers.

**!** **FITTING (Refer to Fig.3) Tools**  Required: 5mm Hexagon Key

- NOTES:**
1. The Adductors are handed by virtue of the central pad clamp position.
  2. It is recommended that the installer fits one side at a time.
- Locate a suitable position on the upper area of each footrest hanger to attach the Mounting Clamps. (**Do Not** attach on a bend).

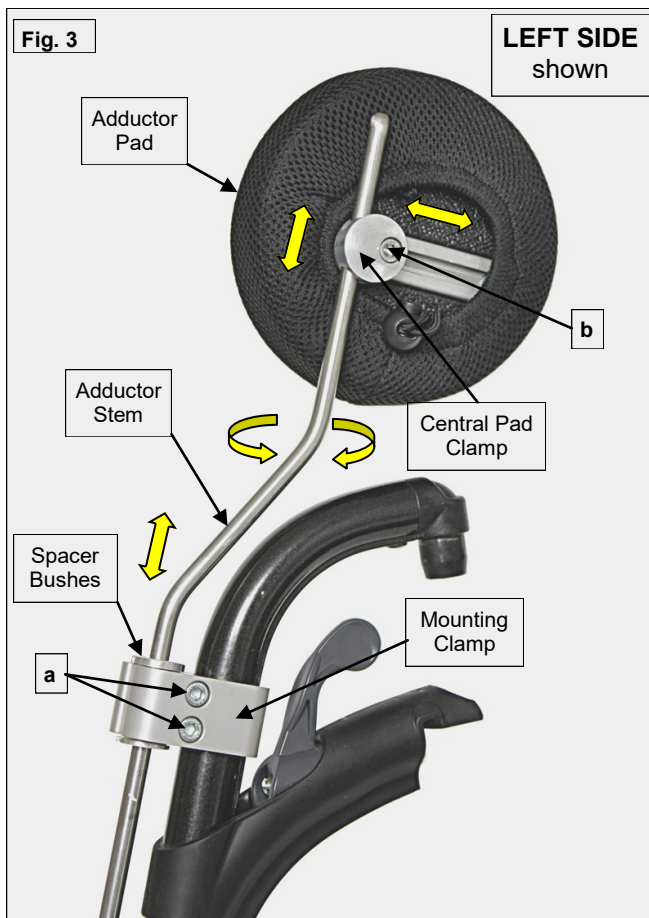

- Separate each Mounting Clamp by removing clamping screws (a) and install onto the footrest hanger. (For ease of later adjustment, the hexagon drive screw heads, should be facing towards the outside of the wheelchair).

- Slide the appropriate side Adductor Stem, complete with Pad, downwards into the Mounting Clamp Spacer Bushes and temporarily secure sufficient to maintain the assembly's position.

- Repeat above process for the opposite side.

## ADJUSTMENT

- With the User seated and correctly positioned in the wheelchair, raise or lower the Adductor Stem in its Mounting Clamp to get the approximate Adductor Pad height required.
- Correct Pad contact with the User's knee can be achieved by rotating the Mounting Clamp and /or Adductor Stem around the footrest hanger.
- Final height, depth or any angle positioning, can be set by slackening screw (b) and moving the Pad to the required position.
- Fully tighten all clamping screws sufficient to prevent any movement.
- Repeat adjustment process for opposite side.


[PTO]

## CLEANING


- Always sponge-off any spillage or contamination as soon as possible, with warm water and soap. The Adductor Pads are fitted with removable covers. These can be wiped over with hot water and soap. A mild hypochlorite solution (Milton) can also be used if preferred. DO NOT use PHENOL based products on any waterproof coverings. The covers should be left to thoroughly air dry before refitting.



Wash at 40°C

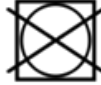

Do not  
tumble dry

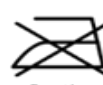

Do not iron



### IMPORTANT NOTE:

Gel-type Adductor Pads may be coated in talcum powder.



### MAINTENANCE

- The Adductors should be checked periodically for security, damage, correct positioning any defects should be reported to the appropriate authority immediately.

**Should the installer require further information on this, or any other product in the RMS Range, please contact our Technical Help-line on 01795 477280**