



**Thompson House**

Unit 10 Styles Close, Sittingbourne, Kent, ME10 3BF

Tel: 01795 477280 Fax: 01795 229692

E-mail address: [sales@rms-kent.co.uk](mailto:sales@rms-kent.co.uk)

[www.ineedawheelchair.co.uk](http://www.ineedawheelchair.co.uk)

EN

MD

## **RMS Leg Abductor Straps**

**For Part Numbers LAS-001, LAS-002 and LAS-003**



RMS Leg Abductor Straps are intended for use as a method of holding each inner thigh and providing resistance to pelvic obliquity and rotation.

The free ends should be attached in a suitable position to the wheelchair frame using the clamps supplied.

After attaching to the wheelchair frame, and with the user seated, route the lower strap under the user's thigh and the upper strap over the thigh. Connect the strap at the pull-tight buckle.

Adjustment is made by pulling as necessary on the "D" ring. To avoid circulation problems, care should be taken not to over tighten these straps, when adjusting using the pull tight buckle.

- Care should be taken not to over-tighten Foot-Straps when adjusting.
- Footstraps should be checked before use for security and damage.

### **Cleaning**

Footstraps can be sponge cleaned using warm water and a mild detergent. Ensure the footstrap is fully dry before use.

**Should you require further information regarding these or any other products in the RMS range, please contact the RMS Technical Help-line on 01795 477280.**



Dec 2021

LAS-IFU-2021-rev0

