




## Thompson House

Unit 10 Styles Close, Sittingbourne, Kent, ME10 3BF

Tel: 01795 477280 Fax: 01795 229692

E-mail address: [sales@rms-kent.co.uk](mailto:sales@rms-kent.co.uk)

[www.ineedawheelchair.co.uk](http://www.ineedawheelchair.co.uk)

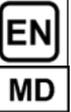

# FOOT-STRAPS

(Covering Part Numbers FS003 S & L)  
**FITTING GUIDELINES**


## NOTES:

FS003 Foot-Straps are intended for use as either Ankle or Toe Straps or both and are suitable for use with Footplates or Footrest Panels having slots or holes to take 25mm nylon webbing.

(See examples below)

The overall length of the Short Foot-Strap (FS 003S) is 450mm and suitable for circumference range of 300 to 370mm.

The overall length of the Long Foot-Strap (FS 003L) is 500mm and suitable for circumference range of 350 to 420mm.

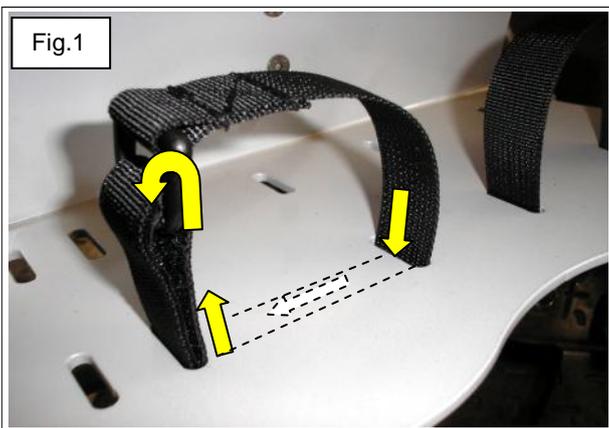


## FITTING (Refer to Figs. 1 & 2)

- With the hook and loop panels of the Foot-Straps facing towards the centre of the wheelchair, feed the free end of each Foot-Strap downwards through the appropriate slot in the Foot Plate/Panel.
- Route the Foot-Strap, across underneath the Foot Plate/Panel, to the opposite appropriate slot, then feed the Foot-Strap end upwards through the slot, (the hook and loop panels should now be facing the outer edge of the Foot Plate/Panel).

- Feed the free end of each Foot-Strap through the “D” ring and secure by folding back onto the hook and loop panels.

- Care should be taken not to over-tighten Foot-Straps when adjusting.
- Footstraps should be checked before use for security and damage.

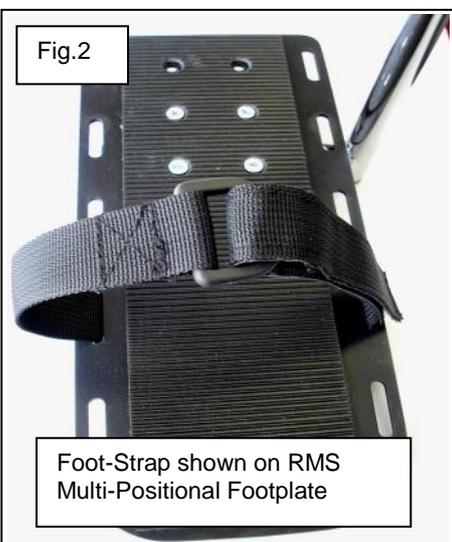

## Cleaning

Footstraps can be sponge cleaned using warm water and a mild detergent. Ensure the footstrap is fully dry before use.



Dec 2021

FS003-IFU-2021-rev0

