

Ventilator / Accessory Trays to suit Otto Bock Discovery Wheelbase

36cm

40cm

45cm

50cm

IMPORTANT NOTES

- The Ventilator / Accessory Tray supplied with these instructions, Fig.1, has been supplied specifically for use with the Otto Bock Discovery wheelbase size indicated above and Installation will be the same for all sizes.
- As no tests have been carried out with this device to confirm its suitability for use during transportation in a motor vehicle, it is recommended, that unless absolutely essential, any equipment normally carried on this Tray, be removed and securely stored for the duration of the journey.
Where removal of equipment is not possible, a suitable risk assessment should be carried out by a suitably qualified person, covering this aspect.
The maximum recommended carrying weight is 10Kg.

TOOLS REQUIRED

- 4mm Hexagon Key

INSTALLATION

The Tray is attached to the Discovery Wheelbase by means of two Clamps at the front and two at the rear, Fig.2.

Two hole positions are available at each mounting point which enables the Tray to be mounted further forwards or rearwards by 50mm.

- Open up each Clamp in turn, installing two onto the Front Cross Frame and two onto the Rear Axle Beam. **NOTE:** A captive Nylock nut is fitted into one side of the Clamps and this should be positioned at the lower side of the Clamp.
- Position the Tray over the Front Cross Frame and Rear Axle Beam and align the holes in the Tray with the Clamps.
- Install the Clamping Screws through the Tray from the top and screw downwards through the Clamps into the Nylock nuts. **DO NOT FULLY TIGHTEN AT THIS STAGE.**
- Centralise the Tray on the wheelbase before fully tightening the screws sufficient to prevent any movement.

CLEANING

The Tray may be wiped with a damp cloth and warm soapy water with a little disinfectant if required.

