




Thompson House
 Unit 10 Styles Close, Sittingbourne, Kent, ME10 3BF
 Tel: 01795 477280 Fax: 01795 229692
 E-mail address: sales@rms-kent.co.uk
www.ineedawheelchair.co.uk

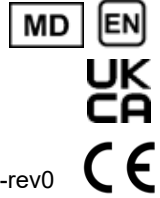


DEC 2021

MARTRAY-IFU-2021-rev0

Marley Accessories

Contour Tray


Fitting Guidelines

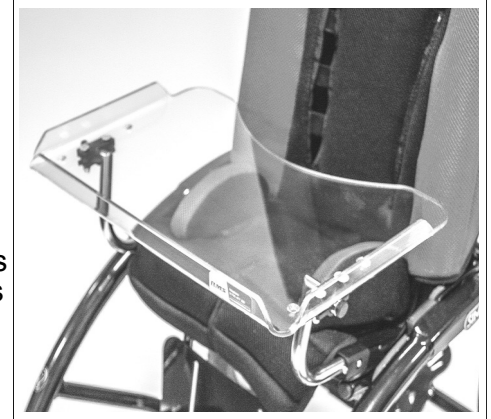

Designed specifically for use with the **Marley** Buggy and available in two sizes to suit either large or small models, the Tray kit comprises of a depth adjustable Contour Tray, Support Stems, Receiver Mounts with fixing screws.




Tools required:

4mm Hexagon Key and 10mm Spanner.

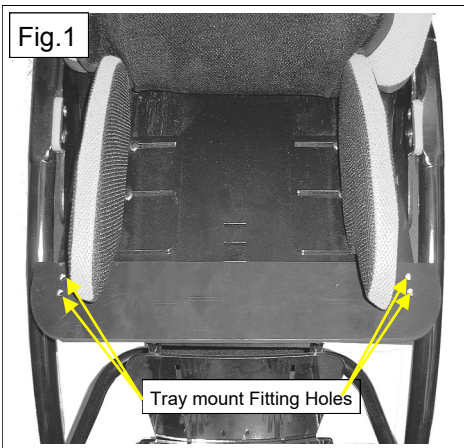

FITTING

- Using the screws supplied, attach one Tray Receiver Mount to the upper face on each side of the Upper Seat Panel, but do not fully tighten at this stage. Fig.2.
- Install the Tray by inserting both Tray Support Frames into their Receiver Mounts by depressing the spring pins (a) Fig.3, which will allow the locking spring pins (b) to engage in the Receiver Mounts. Releasing the spring pins (a) to allow spring pins (b) to engage into their locating holes in the side of the Receiver Mounts.
- With the Tray correctly inserted, fully tighten all four Receiver Mount screws sufficient to prevent any movement.
- To remove the Tray, fully depress both spring pins (a) and slide Tray forwards away from the buggy.
- The angle of Tray can be adjusted by slackening both nuts on each Tray Frame Receiver (c) Fig.3. Position Tray as required and re-tighten all retighten all nuts, sufficient to prevent any movement.
- The Tray depth position can be adjusted by moving the Tray Frame Clamps either forward or rearwards using the pre-drilled holes in the Tray. **NOTE: Tray Frames must be positioned equally on each side.**

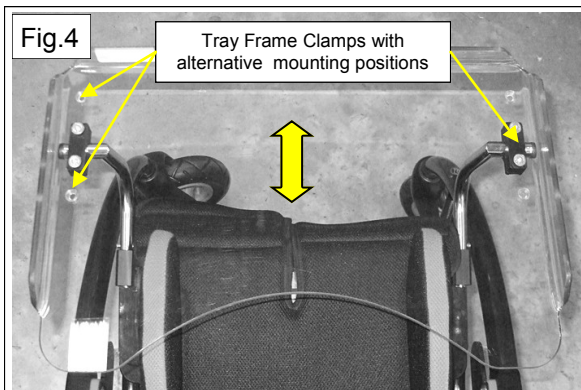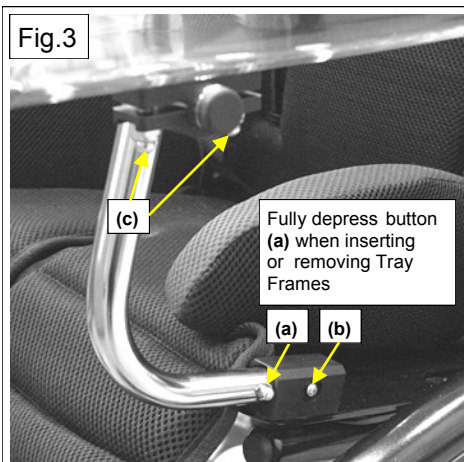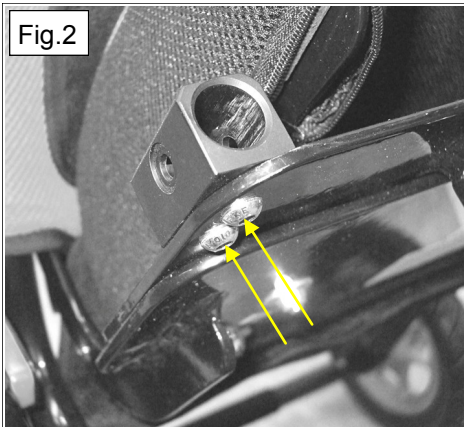

DO NOT OVERTIGHTEN the clamping screws and nuts. **Tighten just sufficient to prevent any movement.** **Inspect** the tray and brackets regularly for position and security. **Do not** place heavy objects on the tray

Should the installer require further information about this, or any other products in the RMS range, please contact the RMS Technical Helpline on 01795 477280.