


Marley Vent Tray

Installation Guidelines



- **Under no circumstances** should the frame webbing straps of the Marley be removed.
- Please ensure the vent tray mounting points are in their correct positions on the Marley's frame before adding any items to the vent tray.
- It may be necessary to perform a suitable stability test prior to issue as heavy items in the vent tray could alter the Marley's stability.
- Items will need to be removed from the vent tray to allow fitment and removal .
- Ensure all equipment is placed in a safe and operable position in the vent tray i.e. kept away from moving parts.
- If using an Oxygen cylinder, ensure it is fitted correctly in the holder, safety strap engaged and that any regulator and associated pipework are in a safe operable position i.e. away from moving parts.
- Regularly inspect the Vent tray for damage, please report any damage immediately to your provider.

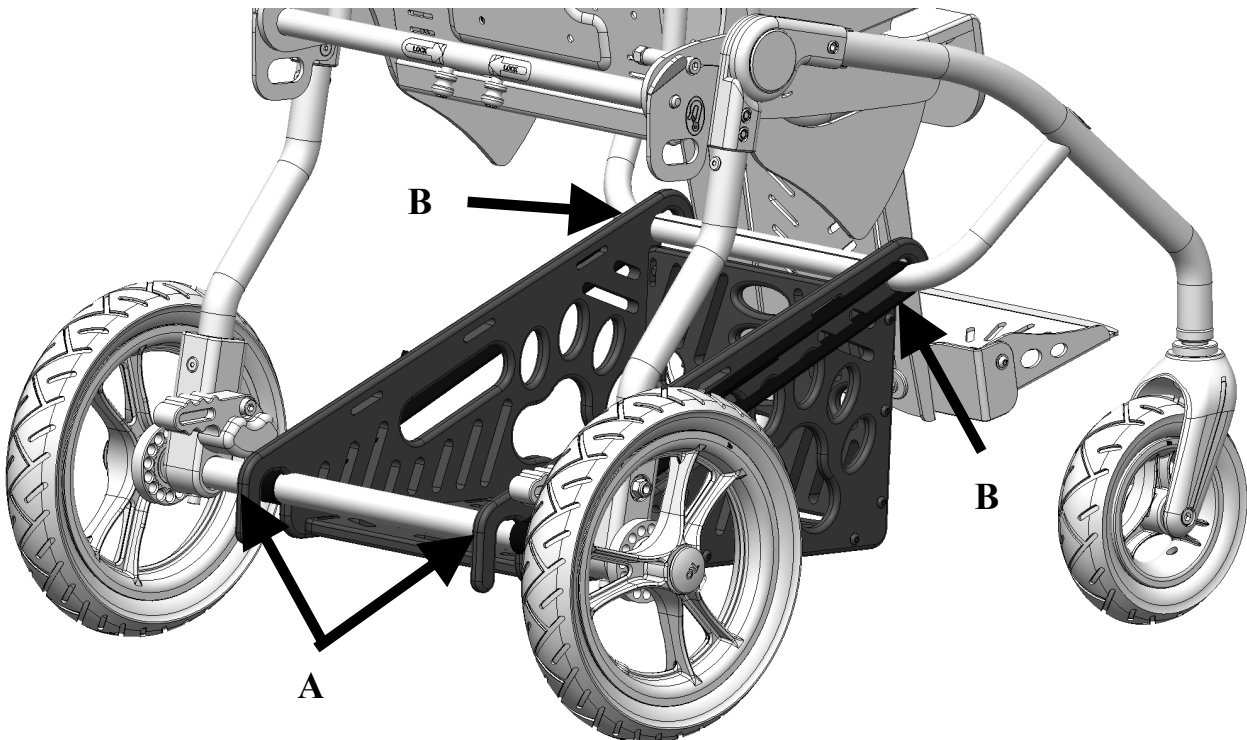

Fitting Instructions

- From the rear of the Marley, place the vent tray in between the frame webbing straps and push forward.
- Slightly tilting the vent tray, slowly lower into place ensuring the rear mounting hooks (A) fit onto the Marley rear axle frame and then push the front down so the vent tray front mounting hooks (B) sit over the front Marley cross member.
- Ensure the vent tray is firmly in place before placing any items onto it.
- In order to allow the marley to fold the Vent tray must first be removed
- Removal is the reverse of the fitment procedure above.



Thompson House, Unit 10 Styles Close,
Sittingbourne, Kent, ME10 3BF
Tel: 01795 477280 Fax: 01795 229692
E-mail address: sales@rms-kent.co.uk
www.inedawheelchair.co.uk



FM 661078

Dec 2021



MARVENT-IFU-2021-rev0