



Thompson House

Unit 10 Styles Close, Sittingbourne, Kent, ME10 3BF

Tel: 01795 477280 Fax: 01795 229692

E-mail address: sales@rms-kent.co.uk
www.ineedawheelchair.co.uk



DEC 2021



MARGRAB-IFU-2021-rev1


Marley Accessories

Grab-Bar for (Marley Sizes 1 & 2)

Fitting Guidelines

Designed specifically for use with the **Marley** Buggy and available in two sizes to suit either large or small models, the Grab-Bar kit comprises of a depth adjustable Grab-Bar and Receiver Mounts with fixing screws.


Tools required: 4mm Hexagon Key.

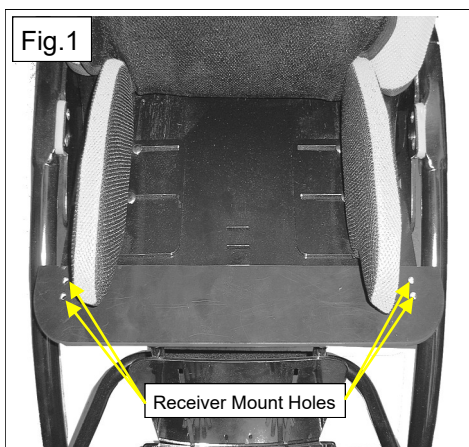

FITTING

- Using the screws supplied, attach one Grab-Bar Receiver Mount to the upper face on each side of the Upper Seat Panel, but do not fully tighten at this stage. Fig.2.



- Install the Grab-Bar by inserting both Grab-Bar Stems into their Receiver Mounts by depressing the spring pins (a) Fig.3, which will allow the locking spring pins (b) to enter the Receiver Mounts. Releasing the spring pins (a) will allow spring pins (b) to engage into their locating holes in the side of the Receiver Mounts.

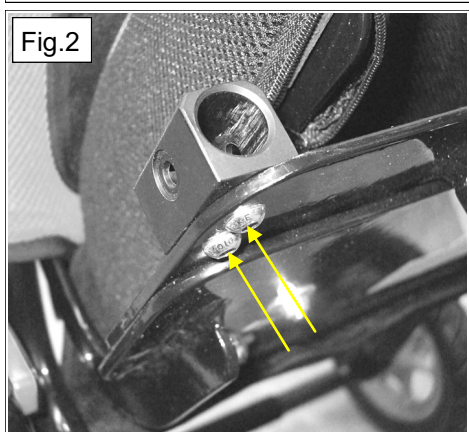

- With the Grab-Bar correctly inserted, fully tighten all four Receiver Mount screws sufficient to prevent any movement.

- To remove the Grab-Bar, fully depress both spring pins (a) and slide Grab-Bar forwards away from the buggy.

- The Grab-Bar can be adjusted forwards or rearwards by depressing the buttons (c) located on each side of the Grab-Bar Frame Fig.4. Position as required and release buttons (c)

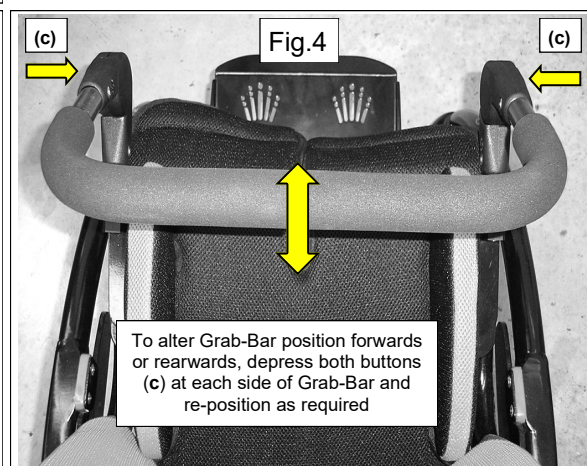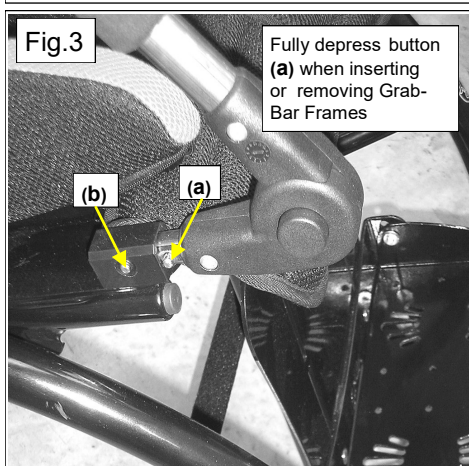

Should the installer require further information about this, or any other products in the RMS range, please contact the RMS Technical Helpline on 01795 477280.