

Marley Canopy



Installation Guidelines

EN

MD



- Do not put items on the canopy when open or folded.
- Do not apply excessive force when opening and closing the canopy.
- Regularly inspect the canopy and mounts for damage and security.
- Remove Canopy before folding the Marley buggy.
- 3mm allen key and 8mm spanner.

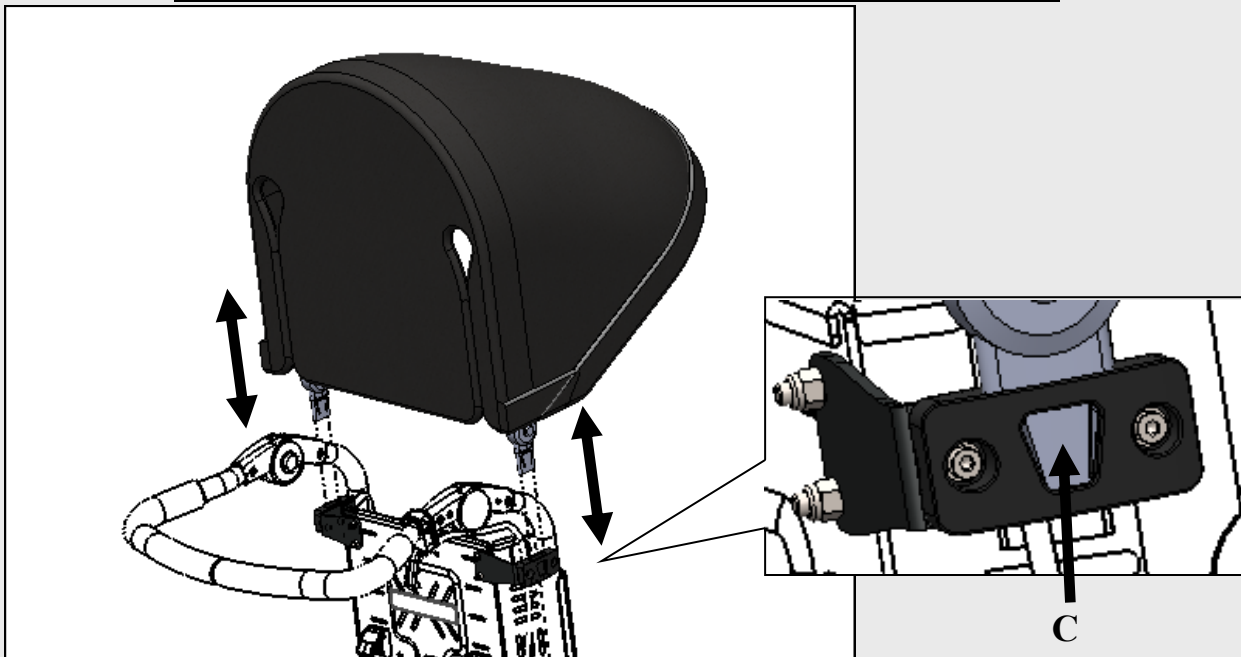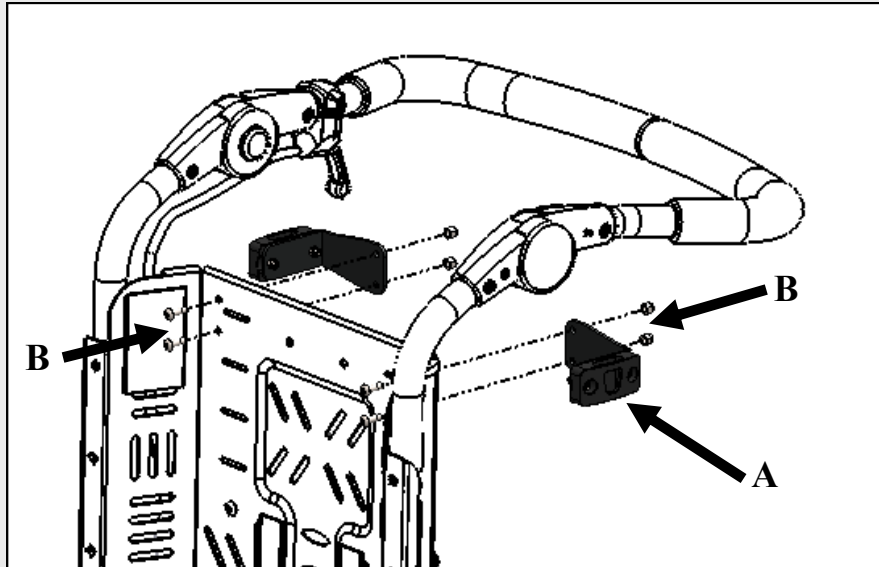

Fitting Instructions

- Attach the mounting brackets **A** to the mounting holes at the rear of the Marley back rest using the screws and nuts **B** provided.
- Insert the canopy locating arms into the mounting brackets until tab **C** clicks into place.
- Check alignment and angle of the canopy is as required.
- To remove depress the centre tab **C** on the locating arm and carefully lift upwards.

Marley Rain Cover

Installation Guidelines



Slip top of rain cover over the canopy and pull around canopy.

Secure the hook and loop straps around the top of the rear frame leg.

Bottom of rain cover slips over the footrest.



Thompson House, Unit 10 Styles Close,
Sittingbourne, Kent, ME10 3BF
Tel: 01795 477280 Fax: 01795 229692
E-mail address: sales@rms-kent.co.uk
www.ineedawheelchair.co.uk



Dec 2021

MARCAN-IFU-2021-rev1

