

# HARNES MOUNTING FRAME



A device to provide upper Harness mounting points, utilising the standard 13mm square Headrest Stem. Also available for 20mm square stems to suit Ormesa Juditta models and 47mm wide rectangular stems to suit Handicare Ibis models.




## FITTING GUIDELINES

Tools Required ; 3, 4 & 5mm Hex.Keys

### INSTALLATION

**RMS recommend that the Harness Mounting Frame and harness are fitted and adjusted by a suitably qualified person.**

- To assist with the installation, the Mounting Frame is supplied loosely assembled. Where possible, the Headrest, complete with it's stem, should be released and removed from the wheelchair.
- Remove height positioning collar if applicable.
- Install the Mounting Frame onto the Headrest stem from the bottom by sliding it upwards. (Clamping Block (a) Fig.1 should be at the front of the stem but do not fully tighten clamping screws at this stage). Refit height positioning collar onto stem.
- Re-install the Headrest, together with the new Mounting Frame attached, into it's receiver and secure at the original position.
- Position the Cross Frame (b) at the required height and secure, by evenly tightening both screws in Clamping Block (a) sufficient to prevent any movement.
- Position the Harness Mounting Clamps, with their Blocks (c) Fig.1, at the required width (i.e. In line with the intended direction of pull from the upper Harness Straps) and secure by tightening their screws sufficient to prevent any movement.
- The upper Harness Straps can now be inserted through the Mounting Clamps and secured at the correct length by folding over and *fully* closing the Mounting Clamp covers.
- The lower Harness Straps should be attached to designated or suitable mounting points on the wheelchair or seating unit.

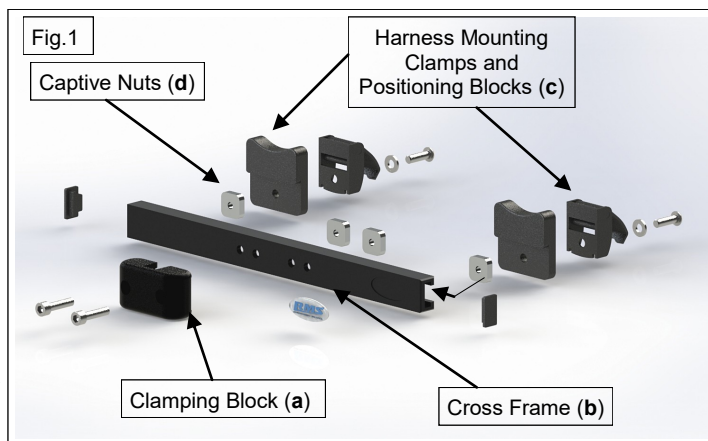

**Regularly inspect to ensure the headrest is correctly secured to its mounting bracket on the wheelchair and that the Harness Mounting Bracket is correctly secured to the headrest stem.**

**Regularly inspect the Harness Mounting Bracket and report any damage to the appropriate authority.**

**Do not hang items from the Mounting frame as this could cause stability issues with the wheelchair.**

Should the installer have any query regarding this installation, please contact :- The RMS Technical Helpline on 01795-477280 or email: [technical@rms-kent.co.uk](mailto:technical@rms-kent.co.uk)



**Thompson House**

Unit 10 Styles Close, Sittingbourne, Kent, ME10 3BF

Tel: 01795 477280 Fax: 01795 229692

E-mail address: [sales@rms-kent.co.uk](mailto:sales@rms-kent.co.uk)

[www.ineedawheelchair.co.uk](http://www.ineedawheelchair.co.uk)



FM 661078

DEC 2021

HMF-IFU-2021-rev0

