


# Special Seating Headrest Mount


MD EN


Designed to accept both 13 and 16mm square headrest stems. Special mounting bracket option allows fitment to backrest shell or channel back systems e.g. V-Trak with or without harness mount.  
Part Numbers: L4-P44, L4-P44-SSHM, L4-P44-SSHMM

## FITTING GUIDELINES: Healthcare Professional

 **Tools Required;** 4mm Hex.Keys, Drill, 6mm drill bit, Countersink bit, torque wrench, 4mm hex bits


### INSTALLATION

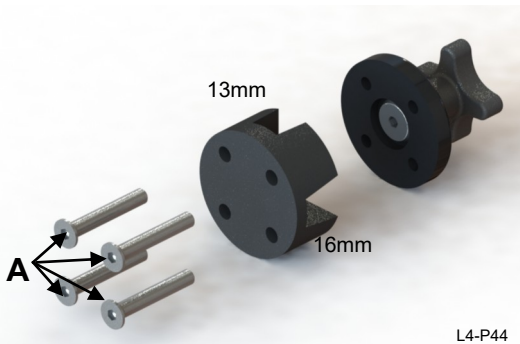

 RMS recommend that the headrest mount is fitted by a **Healthcare Professional**



To fit the headrest mount directly to a backrest panel:

- 1) Remove the M6 hex screws "A"
- 2) Rotate mount to align either 13 or 16mm receiver hole
- 3) Line up the mount with holes on backrest panel; should holes need to be drilled in the backrest panel, hole centres are set at 24mm for 6mm screws and countersunk.
- 4) Insert screws "A" through the backrest panel and mount body, align with threaded holes in the top of the mount and tighten to 5-10Nm

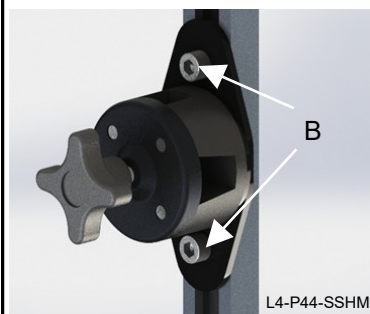


To fit the headrest mount to a channel backrest:

- 1) Ensure the 13 or 16 mm receiver hole is aligned correctly with the fixing plate and tighten screws "A" to 5Nm
- 2) Slacken screws "B" whilst retaining the square nuts "C"
- 3) Insert the square nuts "C" into the backrest channel, once in position tighten screws "B" to 5Nm

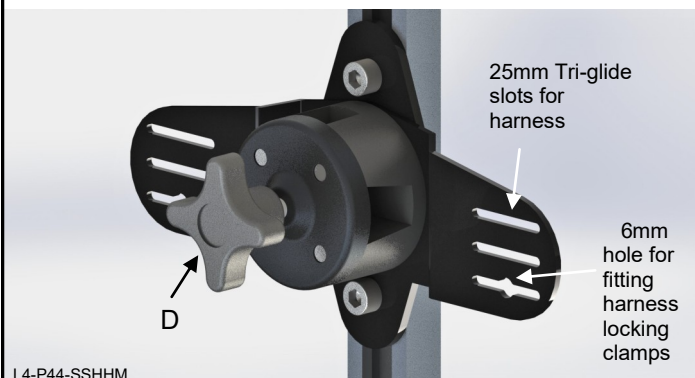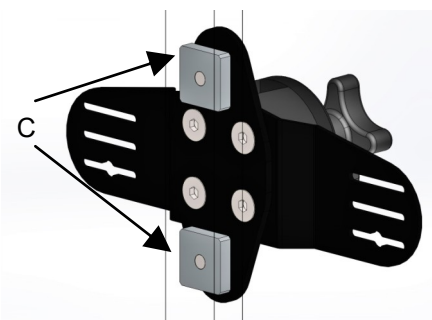





To fit a headrest stem, loosen thumbscrew "D", insert stem to required position and retighten "D".

The optional harness mounting plate allows the attachment of 25mm harness webbing straps to be threaded through the three slots to lock into position or the fitment of a "cam-lock" style webbing locking clamp using the 6mm hole provided.

 Headrest positioning and adjustment should be made by a Healthcare Professional.

### Information: All Users

-  Inspect headrest and harness security and correct adjustment before use.
-  Report any damage immediately to the appropriate authority.
-  **Please inform RMS Ltd of any adverse incidents involving this product and the National Competent authority (MHRA inUK).**
-  Disposal: All parts can be broken down into material types. Please check your local recycling centre for more details.



**Thompson House**

Unit 10 Styles Close, Sittingbourne, Kent, ME10 3BF

Tel: 01795 477280 Fax: 01795 229692

E-mail address: [sales@rms-kent.co.uk](mailto:sales@rms-kent.co.uk)

[www.ineedawheelchair.co.uk](http://www.ineedawheelchair.co.uk)



Dec 2021

SSHMM-IFU-2021-rev 0

